

## PRESTAN FEMALE SERIES 2000 MANIKIN

**SKU:** PP-AFM-2000-1-MS

The same high quality, realistic Professional Female Manikin is now available with Advanced Feedback including Rate, Depth, Recoil, Ventilation, Hands-Off Time, and Chest Compression Fraction via a Bluetooth® enabled PRESTAN CPR Feedback App available on both Apple and Android devices.



## PRESTAN PROFESSIONAL FEMALE SERIES 2000 MANIKIN

The same high quality, realistic Professional Female Manikin is now available with Advanced Feedback including Rate, Depth, Recoil, Ventilation, Hands-Off Time, and Chest Compression Fraction via a Bluetooth® enabled PRESTAN CPR Feedback App available on both Apple and Android devices.

Fully compliant with current industry guidelines, including the Heart and Stroke Foundation of Canada's February 2021 Use of Feedback Devices in Resuscitation Training Courses Policy and the most recent AHA Integrated Feedback Directive.

Available in medium or dark skin tones and in Single or 4-Pack configurations.

All PRESTAN CPR Training Manikins are Made in the USA.

<https://youtu.be/7BD8vKb-CLc>

---

## ADDITIONAL DETAILS:





***\*REDESIGNED\** PRESTAN CPR Feedback App**

Provides high quality, real time, advanced CPR feedback via a Bluetooth® connection with the ***\*NEW & IMPROVED\**** PRESTAN Professional Adult Series 2000 Manikin.



Advanced CPR feedback (includes rate, depth, recoil, ventilation, hands-off time, and ***chest compression fraction***) is monitored, easily displayed and available for reporting allowing students to self-monitor performance and gain confidence during CPR training.

The app allows instructors to provide comprehensive CPR training while monitoring CPR performance for up to six (6) students at a time and is equipped with a reporting feature that allows for CPR training results to be downloaded directly from the app.

The ***\*REDESIGNED\**** PRESTAN CPR Feedback App provides an

- ***improved user experience***
- ***easier setup*** - with no need to calibrate breaths
- more ***intuitive CPR feedback*** - with the addition of chest compression fraction
- ***expanded list of available languages***

Languages currently supported by the PRESTAN CPR Feedback App include German, English (Intl), English (US), Spanish, French, Italian, Dutch, Russian, Korean, Arabic, Malay, Thai, Vietnamese, Belarusian, Croatian, Czech, Danish, Finnish, Greek, Hungarian, Norwegian, Polish, Portuguese, Romanian, Slovak, Slovene, Turkish, Ukrainian, and Chinese.

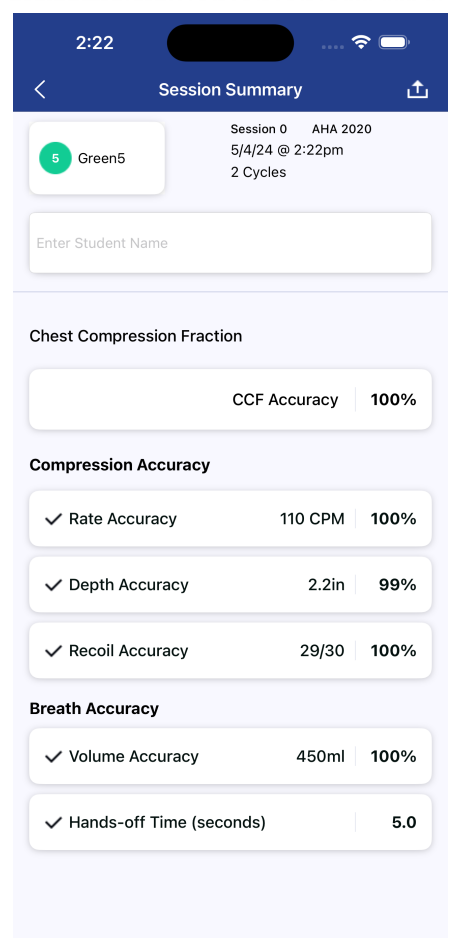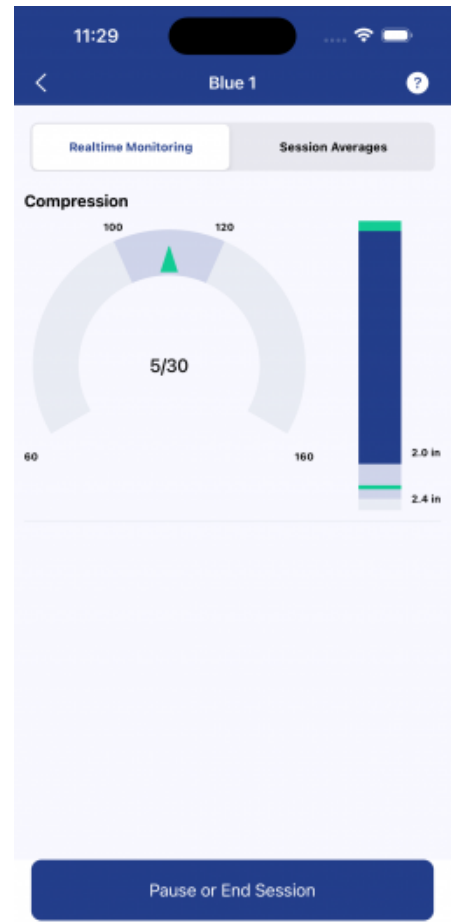

Download the **\*REDESIGNED\*** PRESTAN CPR Feedback App by Scanning the QR Code Below



The PRESTAN CPR Feedback App is for Instrumented Directive Feedback Devices (IDFD) that meet the most recent requirements of the American Heart Association and the Heart and Stroke Foundation of Canada's February 2021 *Use of Feedback Devices in Resuscitation Training Courses Policy*.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation. FCC ID# QOQ-BGM220S2 / FCC ID# 2ASY3AK2G-PA

**\*NEW\* PRESTAN Professional Adult Series 2000 Manikin CPR Rate Monitor LED Sequence**

	Connected Mode	Disconnected Mode
<b>Sleep Mode/No Power</b>		
<b>Smart Device Connected Mode</b> <small>(Cycles in sequence left to right)</small>		n/a
<b>1<sup>st</sup> Full Compression, Not Connected Mode</b> <small>(Wake mode/LED Test Mode)</small>	n/a	
<b>CPM* &lt; 60</b>		
<b>CPM* 60 – 79</b>		
<b>CPM* 80 – 99</b>		
<b>CPM* 100 - 120</b>		
<b>CPM* &gt; 120</b>		
<b>Compressions Stopped, Not Connected Mode</b>		
<b>Compressions Stopped, Not Connected Mode Low Battery Mode</b>		
<b>Compressions Stopped, Connected Mode</b>		

\*CPM = Compressions per minute  
 = Flashing

In combination with PRESTAN's traditional CPR feedback light system installed in the shoulder, the Series 2000 manikin offers a unique, dual-purpose training solution providing instructors with an option to train with either advanced or basic feedback.

The visual feedback gives students a real-life feel of delivering 100-120 compressions per minute. Lights continually signal rate progression, confirming compressions are within the 100- 120 cpm range recommended by the Industry

Guidelines. The yellow light will blink if the 120 cpm upper limit is exceeded, warning students to slow down. Instructors can monitor several students quickly and easily.

In addition to the visual feedback, students will also hear an audible beep as the chest is pushed to the appropriate depth of 2.0 -2.4 inches allowing them to experience the true force needed to deliver real life chest compressions.

[Read More](#)



## PRESTAN INFANT SERIES 2000 MANIKIN

**SKU:** PP-IM-2000-1-MS

The same high quality, realistic Professional Infant Manikin is now available with Advanced Feedback including Rate, Depth, Recoil, Ventilation, Hands-Off Time, and Chest Compression Fraction via a Bluetooth® enabled PRESTAN CPR Feedback App available on both Apple and Android devices.



## PRESTAN PROFESSIONAL INFANT SERIES 2000 MANIKIN

The same high quality, realistic Professional Infant Manikin is now available with Advanced Feedback including Rate, Depth, Recoil, Ventilation, Hands-Off Time, and Chest Compression Fraction via a Bluetooth® enabled PRESTAN CPR Feedback App available on both Apple and Android devices.

Fully compliant with current industry guidelines, including the Heart and Stroke Foundation of Canada's February 2021 *Use of Feedback Devices in Resuscitation Training Courses Policy* and the most recent AHA Integrated Feedback Directive.

Available in medium or dark skin tones and in Single or 4-Pack configurations.

All PRESTAN CPR Training Manikins are Made in the USA.

<https://youtu.be/7BD8vKb-CLc>



## ADDITIONAL DETAILS:

### **\*REDESIGNED\* PRESTAN CPR Feedback App**

Provides high quality, real time, advanced CPR feedback via a Bluetooth® connection with the **\*NEW & IMPROVED\*** PRESTAN Professional Adult Series 2000 Manikin.




Advanced CPR feedback (includes rate, depth, recoil, ventilation, hands-off time, and **chest compression fraction**) is monitored, easily displayed and available for reporting allowing students to self-monitor performance and gain confidence during CPR training.

The app allows instructors to provide comprehensive CPR training while monitoring CPR performance for up to six (6) students at a time and is equipped with a reporting feature that allows for CPR training results to be downloaded directly from the app.

The **\*REDESIGNED\*** PRESTAN CPR Feedback App provides an

- **improved user experience**
- **easier setup** - with no need to calibrate breaths
- more **intuitive CPR feedback** - with the addition of chest compression fraction
- **expanded list of available languages**

Languages currently supported by the PRESTAN CPR Feedback App include German, English (Intl), English (US), Spanish, French, Italian, Dutch, Russian, Korean, Arabic, Malay, Thai, Vietnamese, Belarusian, Croatia, Czech, Danish, Finnish, Greek, Hungarian, Norwegian, Polish, Portuguese, Romanian, Slovak, Slovene, Turkish, Ukrainian, and Chinese.

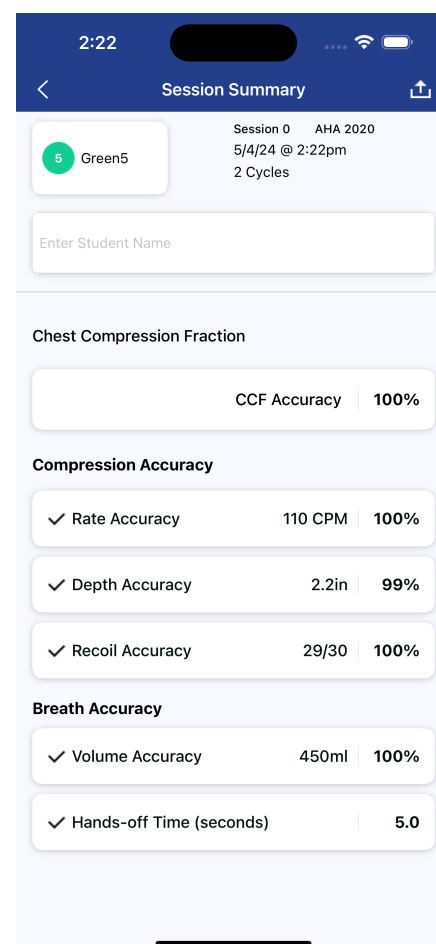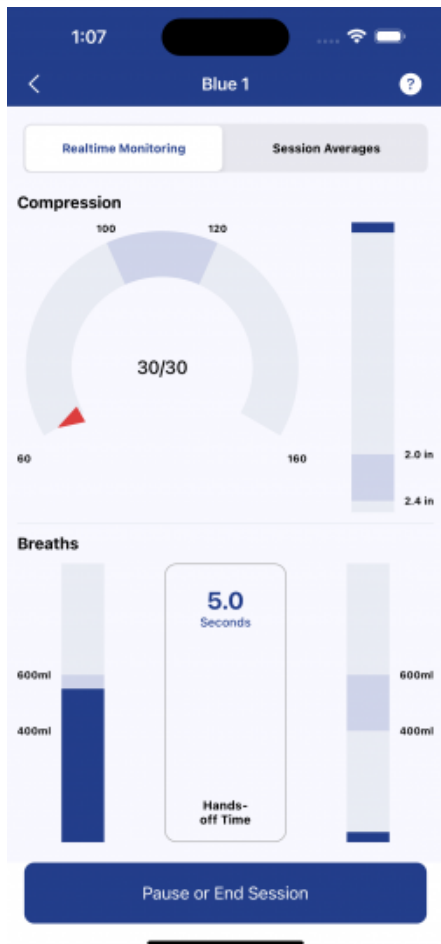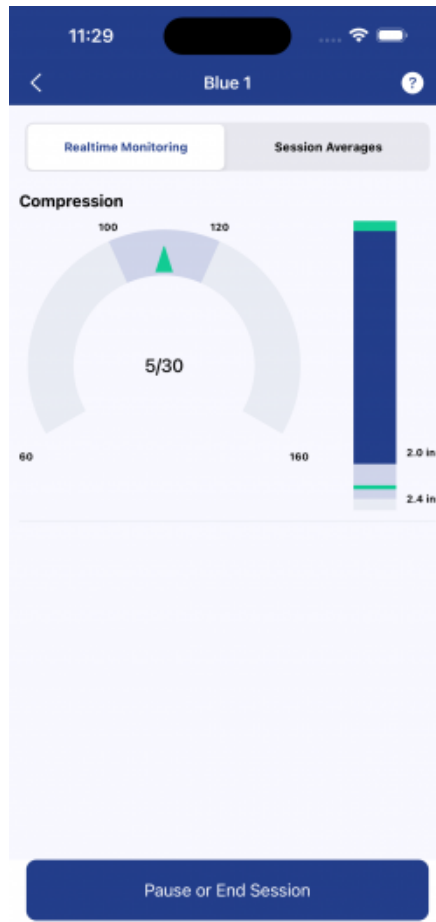

Download the **\*REDESIGNED\*** PRESTAN CPR Feedback App by Scanning the QR Code Below



The PRESTAN CPR Feedback App is for Instrumented Directive Feedback Devices (IDFD) that meet the most recent requirements of the American Heart Association and the Heart and Stroke Foundation of Canada's February 2021 *Use of Feedback Devices in Resuscitation Training Courses Policy*.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation. FCC ID# QOQ-BGM220S2 / FCC ID# 2ASY3AK2G-PA

**\*NEW\* PRESTAN Professional Adult Series 2000 Manikin CPR Rate Monitor LED Sequence**

	Connected Mode	Disconnected Mode
<b>Sleep Mode/No Power</b>		
<b>Smart Device Connected Mode</b> <small>(Cycles in sequence left to right)</small>		n/a
<b>1<sup>st</sup> Full Compression, Not Connected Mode</b> <small>(Wake mode/LED Test Mode)</small>	n/a	
<b>CPM* &lt; 60</b>		
<b>CPM* 60 – 79</b>		
<b>CPM* 80 – 99</b>		
<b>CPM* 100 - 120</b>		
<b>CPM* &gt; 120</b>		
<b>Compressions Stopped, Not Connected Mode</b>		
<b>Compressions Stopped, Not Connected Mode Low Battery Mode</b>		
<b>Compressions Stopped, Connected Mode</b>		

\*CPM = Compressions per minute  
 = Flashing

In combination with PRESTAN's traditional CPR feedback light system installed in the diaper, the Series 2000 manikin offers a unique, dual-purpose training solution providing instructors with an option to train with either advanced or basic feedback.

The visual feedback gives students a real-life feel of delivering 100-120 compressions per minute. Lights continually signal rate progression, confirming compressions are within the 100- 120 cpm range recommended by the Industry

Guidelines. The yellow light will blink if the 120 cpm upper limit is exceeded, warning students to slow down. Instructors can monitor several students quickly and easily.

In addition to the visual feedback, students will also hear an audible beep as the chest is pushed to the appropriate depth of 1.5 inches allowing them to experience the true force needed to deliver real life chest compressions.

[Read More](#)



## PRESTAN CHILD SERIES 2000 MANIKIN

**SKU:** PP-CM-2000-1-MS

The same high quality, realistic Professional Child Manikin is now available with Advanced Feedback including Rate, Depth, Recoil, Ventilation, Hands-Off Time, and Chest Compression Fraction via a Bluetooth® enabled PRESTAN CPR Feedback App available on both Apple and Android devices.



## PRESTAN PROFESSIONAL CHILD SERIES 2000 MANIKIN

The same high quality, realistic Professional Child Manikin is now available with Advanced Feedback including Rate, Depth, Recoil, Ventilation, Hands-Off Time, and Chest Compression Fraction via a Bluetooth® enabled PRESTAN CPR Feedback App available on both Apple and Android devices.

Fully compliant with current industry guidelines, including the Heart and Stroke Foundation of Canada's February 2021 Use of Feedback Devices in Resuscitation Training Courses Policy and the most recent AHA Integrated Feedback Directive.

Available in medium or dark skin tones and in Single or 4-Pack configurations.

All PRESTAN CPR Training Manikins are Made in the USA.

<https://youtu.be/7BD8vKb-CLc>



## ADDITIONAL DETAILS:

### **\*REDESIGNED\* PRESTAN CPR Feedback App**

Provides high quality, real time, advanced CPR feedback via a Bluetooth® connection with the **\*NEW & IMPROVED\*** PRESTAN Professional Adult Series 2000 Manikin.




Advanced CPR feedback (includes rate, depth, recoil, ventilation, hands-off time, and **chest compression fraction**) is monitored, easily displayed and available for reporting allowing students to self-monitor performance and gain confidence during CPR training.

The app allows instructors to provide comprehensive CPR training while monitoring CPR performance for up to six (6) students at a time and is equipped with a reporting feature that allows for CPR training results to be downloaded directly from the app.

The **\*REDESIGNED\*** PRESTAN CPR Feedback App provides an

- **improved user experience**
- **easier setup** - with no need to calibrate breaths
- more **intuitive CPR feedback** - with the addition of chest compression fraction
- **expanded list of available languages**

Languages currently supported by the PRESTAN CPR Feedback App include German, English (Intl), English (US), Spanish, French, Italian, Dutch, Russian, Korean, Arabic, Malay, Thai, Vietnamese, Belarusian, Croatian, Czech, Danish, Finnish, Greek, Hungarian, Norwegian, Polish, Portuguese, Romanian, Slovak, Slovene, Turkish, Ukrainian, and Chinese.

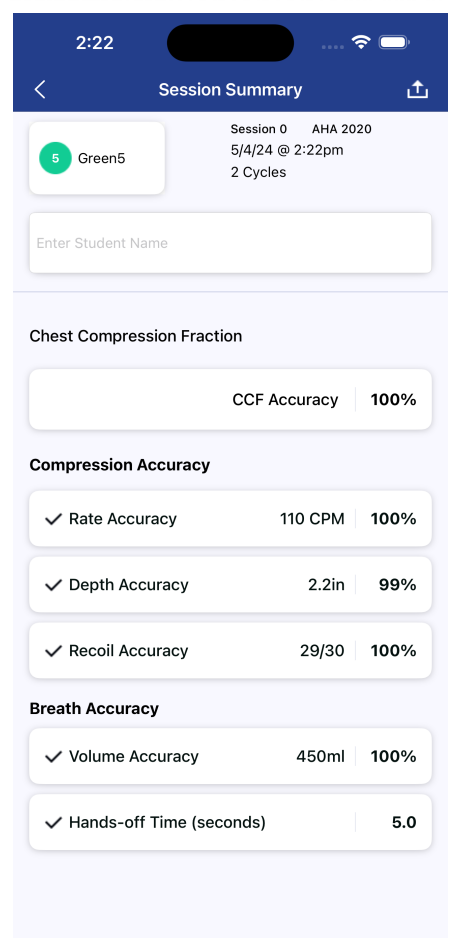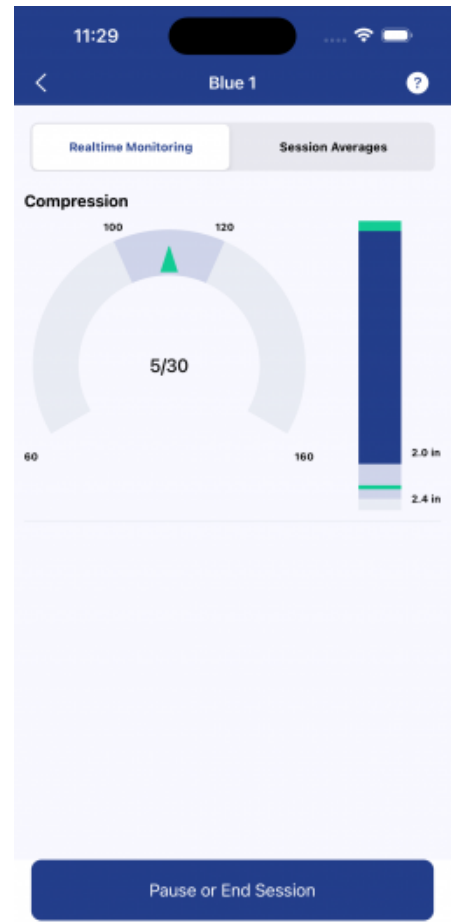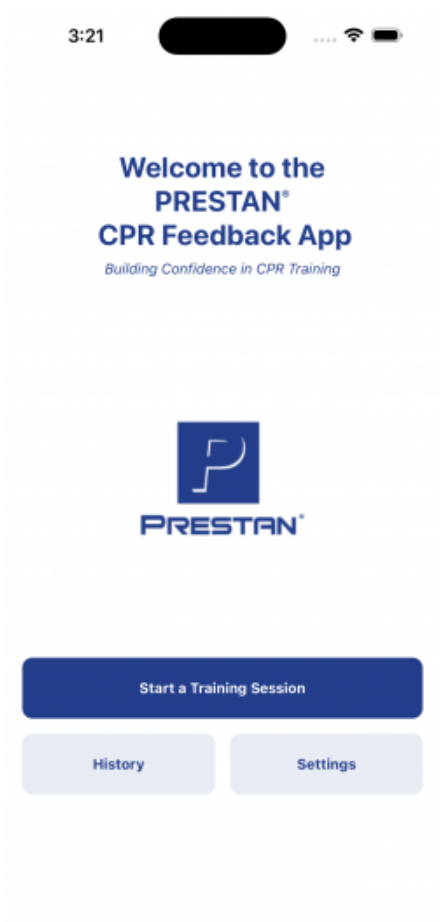

Download the **\*REDESIGNED\*** PRESTAN CPR Feedback App by Scanning the QR Code Below



The PRESTAN CPR Feedback App is for Instrumented Directive Feedback Devices (IDFD) that meet the most recent requirements of the American Heart Association and the Heart and Stroke Foundation of Canada's February 2021 *Use of Feedback Devices in Resuscitation Training Courses Policy*.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation. FCC ID# QOQ-BGM220S2 / FCC ID# 2ASY3AK2G-PA

**\*NEW\* PRESTAN Professional Adult Series 2000 Manikin CPR Rate Monitor LED Sequence**

	Connected Mode	Disconnected Mode
<b>Sleep Mode/No Power</b>		
<b>Smart Device Connected Mode</b> <small>(Cycles in sequence left to right)</small>		n/a
<b>1<sup>st</sup> Full Compression, Not Connected Mode</b> <small>(Wake mode/LED Test Mode)</small>	n/a	
<b>CPM* &lt; 60</b>		
<b>CPM* 60 – 79</b>		
<b>CPM* 80 – 99</b>		
<b>CPM* 100 - 120</b>		
<b>CPM* &gt; 120</b>		
<b>Compressions Stopped, Not Connected Mode</b>		
<b>Compressions Stopped, Not Connected Mode Low Battery Mode</b>		
<b>Compressions Stopped, Connected Mode</b>		

\*CPM = Compressions per minute  
 = Flashing

In combination with PRESTAN's traditional CPR feedback light system installed in the shoulder, the Series 2000 manikin offers a unique, dual-purpose training solution providing instructors with an option to train with either advanced or basic feedback.

The visual feedback gives students a real-life feel of delivering 100-120 compressions per minute. Lights continually signal rate progression, confirming compressions are within the 100- 120 cpm range recommended by the Industry

Guidelines. The yellow light will blink if the 120 cpm upper limit is exceeded, warning students to slow down. Instructors can monitor several students quickly and easily.

In addition to the visual feedback, students will also hear an audible beep as the chest is pushed to the appropriate depth of 2.0 inches allowing them to experience the true force needed to deliver real life chest compressions.

[Read More](#)



## PRESTAN ADULT SERIES 2000 MANIKIN

**SKU:** PP-AM-2000-1-MS

The same high quality, realistic Professional Adult Manikin is now available with Advanced Feedback including Rate, Depth, Recoil, Ventilation, Hands-Off Time, and Chest Compression Fraction via a Bluetooth® enabled PRESTAN CPR Feedback App available on both Apple and Android devices.

## PRESTAN PROFESSIONAL ADULT SERIES 2000 MANIKIN

The same high quality, realistic Professional Adult Manikin is now available with Advanced Feedback including Rate, Depth, Recoil, Ventilation, Hands-Off Time, and Chest Compression Fraction via a Bluetooth® enabled PRESTAN CPR Feedback App available on both Apple and Android devices.

Fully compliant with current industry guidelines, including the Heart and Stroke Foundation of Canada's February 2021 Use of Feedback Devices in Resuscitation Training Courses Policy and the most recent AHA Integrated Feedback Directive.

Available in medium or dark skin tones, with CPR Feedback only and in Single or 4-Pack configurations.

All PRESTAN CPR Training Manikins are Made in the USA.



## ADDITIONAL DETAILS:

### **\*REDESIGNED\* PRESTAN CPR Feedback App**

Provides high quality, real time, advanced CPR feedback via a Bluetooth® connection with the **\*NEW & IMPROVED\*** PRESTAN Professional Adult Series 2000 Manikin.




Advanced CPR feedback (includes rate, depth, recoil, ventilation, hands-off time, and **chest compression fraction**) is monitored, easily displayed and available for reporting allowing students to self-monitor performance and gain confidence during CPR training.

The app allows instructors to provide comprehensive CPR training while monitoring CPR performance for up to six (6) students at a time and is equipped with a reporting feature that allows for CPR training results to be downloaded directly from the app.

The **\*REDESIGNED\*** PRESTAN CPR Feedback App provides an

- **improved user experience**
- **easier setup** - with no need to calibrate breaths
- more **intuitive CPR feedback** - with the addition of chest compression fraction
- **expanded list of available languages**

Languages currently supported by the PRESTAN CPR Feedback App include German, English (Intl), English (US), Spanish, French, Italian, Dutch, Russian, Korean, Arabic, Malay, Thai, Vietnamese, Belarusian, Croation, Czech, Danish, Finnish, Greek, Hungarian, Norwegian, Polish, Portuguese, Romanian, Slovak, Slovene, Turkish, Ukrainian, and Chinese.

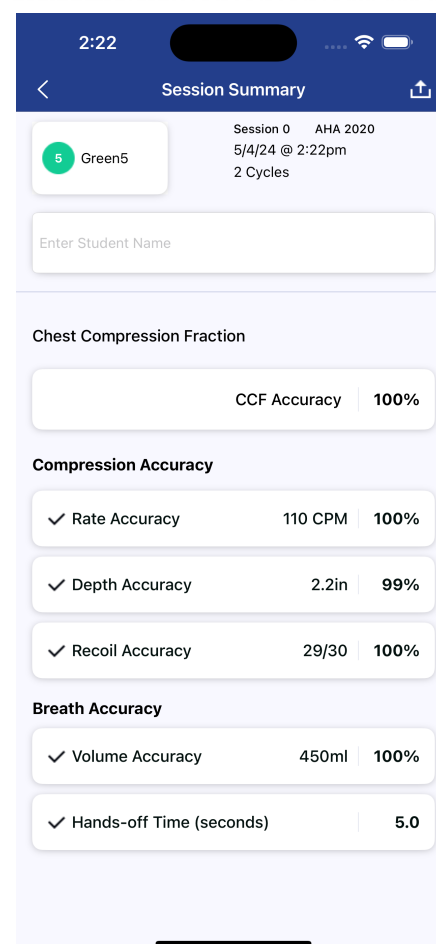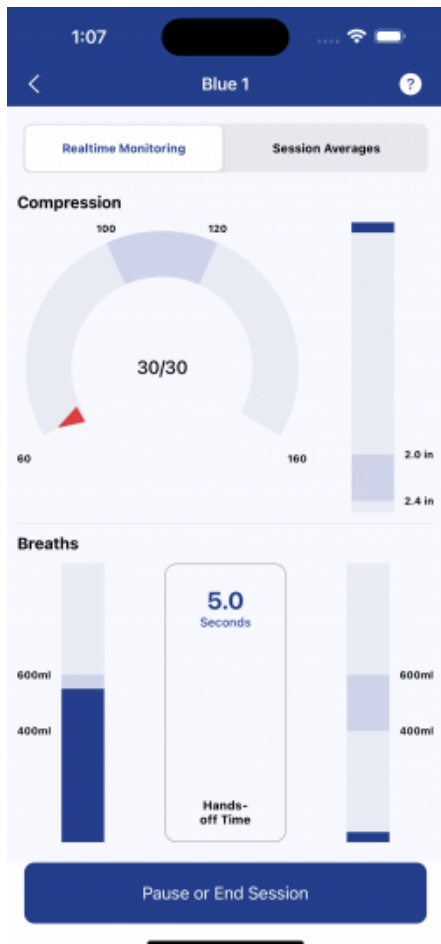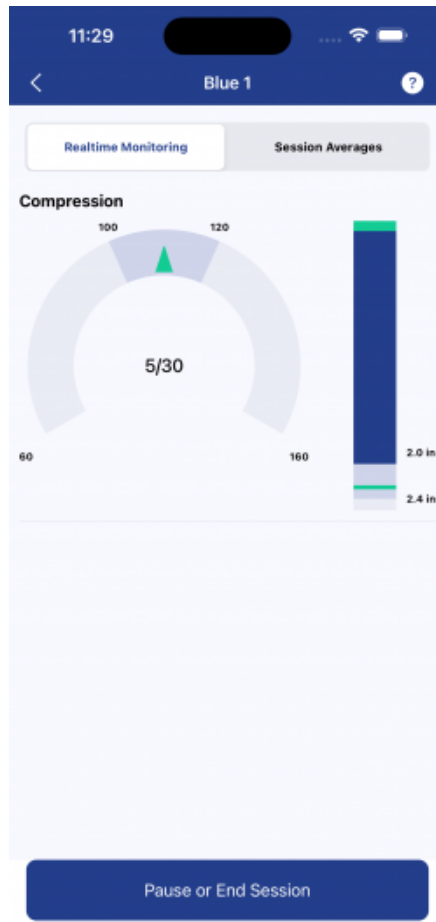

Download the **\*REDESIGNED\*** PRESTAN CPR Feedback App by Scanning the QR Code Below



The PRESTAN CPR Feedback App is for Instrumented Directive Feedback Devices (IDFD) that meet the most recent requirements of the American Heart Association and the Heart and Stroke Foundation of Canada's February 2021 *Use of Feedback Devices in Resuscitation Training Courses Policy*.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation. FCC ID# QOQ-BGM220S2 / FCC ID# 2ASY3AK2G-PA

**\*NEW\* PRESTAN Professional Adult Series 2000 Manikin CPR Rate Monitor LED Sequence**

	Connected Mode	Disconnected Mode
<b>Sleep Mode/No Power</b>	○ ○ ○ ○	
<b>Smart Device Connected Mode</b> <small>(Cycles in sequence left to right)</small>	● ● ● ●	n/a
<b>1<sup>st</sup> Full Compression, Not Connected Mode</b> <small>(Wake mode/LED Test Mode)</small>	n/a	● ● ● ●
<b>CPM* &lt; 60</b>	○ ● ○ ○	
<b>CPM* 60 – 79</b>	○ ○ ● ○	
<b>CPM* 80 – 99</b>	● ○ ○ ○	
<b>CPM* 100 - 120</b>	● ○ ○ ●	
<b>CPM* &gt; 120</b>	● ○ ● ●	
<b>Compressions Stopped, Not Connected Mode</b>	○ ● ○ ○	
<b>Compressions Stopped, Not Connected Mode Low Battery Mode</b>	○ ● ● ○	
<b>Compressions Stopped, Connected Mode</b>	○ ○ ○ ○	

\*CPM = Compressions per minute  
● = Flashing

In combination with PRESTAN's traditional CPR feedback light system installed in the shoulder, the Series 2000 manikin offers a unique, dual-purpose training solution providing instructors with an option to train with either advanced or basic feedback.

The visual feedback gives students a real-life feel of delivering 100-120 compressions per minute. Lights continually signal rate progression, confirming compressions are within the 100- 120 cpm range recommended by the Industry Guidelines. The yellow light will blink if the 120 cpm upper limit is exceeded, warning students to slow down. Instructors can monitor several students quickly and easily.

In addition to the visual feedback, students will also hear an audible beep as the chest is pushed to the appropriate depth of 2.0 -2.4 inches allowing them to experience the true force needed to deliver real life chest compressions.

### Series 2000 Add-On Kit


The PRESTAN Professional Adult Series 2000 Manikin Add-On Kit provides instructors with an option to upgrade their current PRESTAN Professional Adult and Jaw Thrust Manikins with the new Advanced Feedback parameters.

The PRESTAN Adult Series 2000 Add-On Kit is recommended for 2018 manikin models or newer.

**Please Note:** The newest version of the Series 2000 Add-On Kit contains a blue connector attached to the end of the white ventilation cord; however, the installation steps remain the same.

### Series 2000 Lung Bags

The PRESTAN Professional Adult Series 2000 Lung Bags use the same easy to install PRESTAN design with the addition of a breath sensor. The Series 2000 lung bags are required for ventilation feedback via the PRESTAN CPR Feedback App.

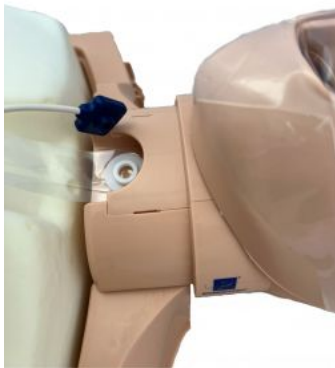

[Read More](#)



## PRESTAN ADULT MANIKINS

**SKU:** PP-AM-100M-MS




The PRESTAN Professional Training Manikin series is unlike any other on the market. Uniquely designed for realistic experiences, they are available individually or in sets of four for group training. CPR manikins are designed to match the real-life CPR experience.

Features:

- Fully compliant with current industry guidelines, including the most recent AHA Integrated Feedback Directive.
- Realistic looking and performing manikin is more acceptable to both instructors and students.
- Far easier and faster to carry and set up at training sites.
- Patented face/head tilt simulates how an actual victim's head would move if he required CPR and teaches students how to properly open the airway.
- Easily inserted face-shield/lung-bag provides visual chest rise to ensure students receive the most accurate and realistic training.
- The face shield is secured so it won't slip and there's no need for adhesives.
- Manikins are available with the CPR Feedback Monitor - Monitors may also be purchased separately to easily install into non-monitored Prestan Professional Manikins.
- Prestan CPR Manikins are not made with natural rubber latex.

Download PDF:

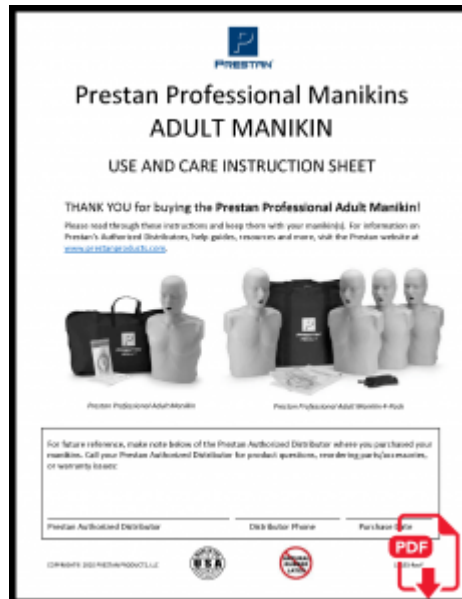

[Read More](#)



## PRESTAN FEMALE MANIKIN

**SKU:** PP-AFM-100M-MS

The PRESTAN Professional Female Manikin is designed to transform CPR/AED training, improve CPR survival rates in women, encourage dialogue & breakdown barriers, and eliminate gender bias in CPR administration.

Using the same trusted design as the Professional Adult Manikin, the Female Manikin is realistic to the eye and the touch, unlike any other CPR Training Manikin on the market. Lightweight with fast and simple setup. Equipped with a revolutionary CPR Rate Monitor that allows for instant rate and depth feedback to both the instructor and student.

Fully compliant with current industry guidelines, including the most recent AHA Integrated Feedback Directive.

Available in medium or dark skin tones, with CPR Feedback only and in Single, 4-Pack or 4-Pack Diversity Kit configurations. Replacement skins are also available and can be used with the PRESTAN Professional Adult, Adult Series 2000, PRO+ Adult, and Adult Jaw Thrust Manikins.

All PRESTAN CPR Training Manikins are Made in the USA.



## ADDITIONAL DETAILS:

### Transform Training with Female CPR/AED Scenarios

When used in combination with the PRESTAN Female Accessory (to represent a bra), Manikin Shirt, and AED Trainer - PRESTAN offers the most realistic training experience to help build confidence when learning these critical lifesaving skills.



### PRESTAN Professional Manikin CPR Rate Monitor

All PRESTAN Professional Manikins are available with a revolutionary CPR Rate Monitor that allows for instant feedback to both instructors and students. The visual feedback from the CPR Rate Monitor gives students a real life feel of delivering 100-120 compressions per minute. Lights continually signal rate progression, confirming compressions are within the 100- 120 cpm range recommended by the Industry Guidelines. The yellow light will blink if the 120 cpm upper limit is exceeded, warning students to slow down. Instructors can monitor several students quickly and easily.


In addition to the visual CPR Rate Monitor, students will also hear a clicker sound as the chest is pushed to the appropriate depth of 2.0 -2.4 inches allowing them to experience the true force needed to deliver real life chest compressions.

The CPR Rate Monitor may be purchased separately to upgrade any non-monitored PRESTAN Professional Manikin.


<b>Sleep Mode/No Power</b>	○ ○ ○ ○
<b>1<sup>st</sup> Full Compression</b> (Wake mode/LED Test Mode)	● ● ● ●
<b>CPM* &lt; 60</b>	○ ● ○ ○
<b>CPM* 60 – 79</b>	○ ○ ● ○
<b>CPM* 80 – 99</b>	● ○ ○ ○
<b>CPM* 100 - 120</b>	● ○ ○ ●
<b>CPM* &gt; 120</b>	● ○ ● ●
<b>Compressions Stopped</b>	○ ● ○ ○

\*CPM = Compressions per minute

⚙ = Flashing

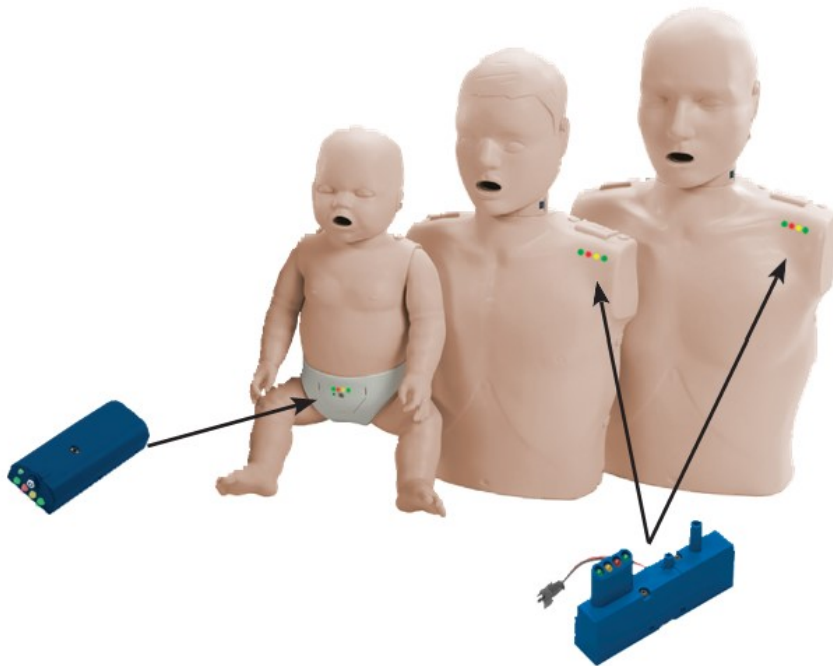

[Download PDF:](#)

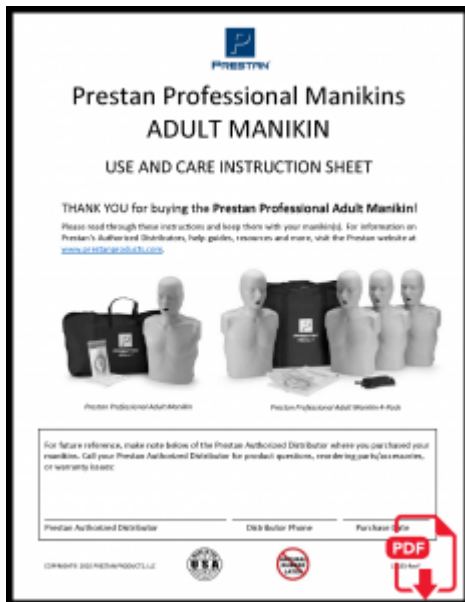


**PRESTAN**

## Prestan Professional Manikins ADULT MANIKIN

### USE AND CARE INSTRUCTION SHEET

**THANK YOU** for buying the **Prestan Professional Adult Manikin!**  
Please read through these instructions and keep them with your manikin(s). For information on Prestan's Authorized Distributors, help, guides, resources and more, visit the Prestan website at [www.prestan.com.sg](http://www.prestan.com.sg)





Prestan Professional ADULT Manikin      Prestan Professional ADULT Manikin P-P02

For future reference, make note below of the Prestan Authorized Distributor where you purchased your manikin. Call your Prestan Authorized Distributor for product questions, service tips, accessories, or warranty issues:

Prestan Authorized Distributor	Distributor Phone	Purchase Date

COMPANY: PRESTAN PRODUCTS LLC



[Read More](#)



## PRESTAN PRO+ ADULT MANIKIN

**SKU:** PP-AMP-100M-MS

Using the same trusted design of the Professional Adult Manikin, PRESTAN is now introducing the PRO+ Adult Manikin featuring a new head design with functional oral and nasal airways along with an improved lung bag design for fast and easy installation.

With the introduction of the new PRO+ Adult Head design, PRESTAN is enhancing realism with a functional nasal airway and the ability to use a NARCAN(R) or NPA training device.

Fully compliant with current industry guidelines, including the most recent AHA Integrated Feedback Directive.

Available in medium or dark skin tones, with CPR Feedback only, in Single or 4-Pack configurations; the Professional Manikin Collection is available in Europe only.


The PRO+ Adult Head is also sold separately, allowing you to upgrade the Professional Adult Manikin.

All PRESTAN CPR Training Manikins are Made in the USA.

## ADDITIONAL DETAILS:

### Designed For Use With a NPA Device

When lubricated properly, a nasopharyngeal airway (NPA) can be inserted through the nose of the PRO+ Adult Head to simulate opening the nasal airway during training.

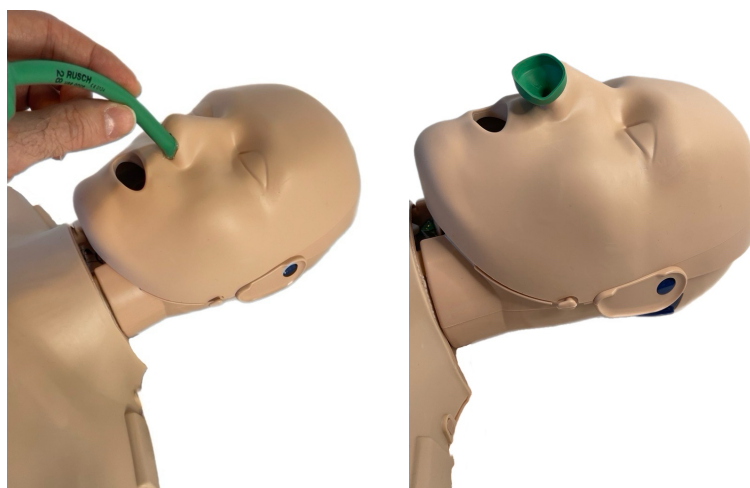

### PRO+ Adult Lung Bags

The new PRO+ Adult Head design allows for a simplified lung bag installation; please refer to the PRO+ Adult Head Help Guide or Sales Video for more information on how to install the lung bag.



### PRO+ Adult Face Shields

A new PRO+ Adult Face Shield has been designed to fully cover both oral and nasal airways; the current PRESTAN Professional Adult Face Shield can also be used.





**PRESTAN Professional Manikin CPR Rate Monitor**

All PRESTAN Professional Manikins are available with a revolutionary CPR Rate Monitor that allows for instant feedback to both instructors and students. The visual feedback from the CPR Rate Monitor gives students a real life feel of delivering 100-120 compressions per minute. Lights continually signal rate progression, confirming compressions are within the 100- 120 cpm range recommended by the Industry Guidelines. The yellow light will blink if the 120 cpm upper limit is exceeded, warning students to slow down. Instructors can monitor several students quickly and easily.


In addition to the visual CPR Rate Monitor, students will also hear a clicker sound as the chest is pushed to the appropriate depth of 2.0 -2.4 inches allowing them to experience the true force needed to deliver real life chest compressions.

The CPR Rate Monitor may be purchased separately to upgrade any non-monitored PRESTAN Professional Manikin.

<b>Sleep Mode/No Power</b>	
<b>1<sup>st</sup> Full Compression</b> (Wake mode/LED Test Mode)	
<b>CPM* &lt; 60</b>	
<b>CPM* 60 – 79</b>	
<b>CPM* 80 – 99</b>	
<b>CPM* 100 - 120</b>	
<b>CPM* &gt; 120</b>	
<b>Compressions Stopped</b>	

\*CPM = Compressions per minute

= Flashing

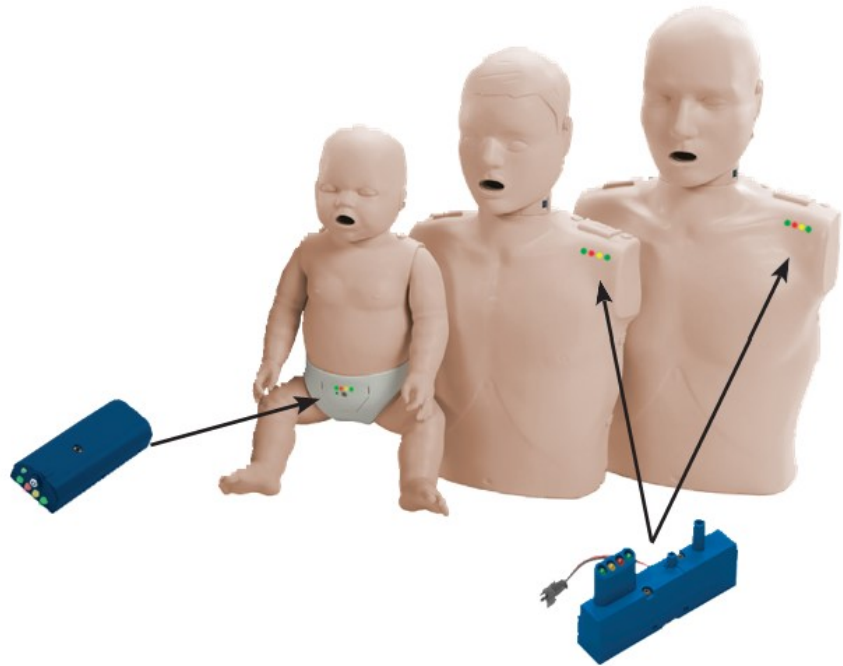

Download PDF:

The image is a thumbnail of a PDF instruction sheet for the Prestan Professional Adult Manikin. It features the Prestan logo at the top, followed by the title "Prestan Professional Manikins ADULT MANIKIN" and "USE AND CARE INSTRUCTION SHEET". Below the title, there is a "THANK YOU" message and a paragraph of text. The middle section shows images of the manikin and its carrying case. At the bottom, there are fields for "Prestan Authorized Distributor", "Distributor Phone", and "Purchase Date", along with a "PDF" icon and a download arrow.

[Read More](#)



## PRESTAN INFANT ULTRALITE MANIKIN

**SKU:** #PP-IULM-100M

As with all PRESTAN manikins, the Infant Ultralite Manikin provides a realistic training experience with anatomically correct features, accurate head tilt and visible chest rise while providing intuitive feedback during CPR training.

Available in convenient and lightweight Single, 4-pack and 12-packs for efficient on-the-go training. This durable manikin offers an affordable solution for CPR training, with all the quality and realism you expect from PRESTAN.

Fully compliant with current industry guidelines, including the most recent AHA Integrated Feedback Directive.

Available in medium or dark skin tones in Single, 4-Pack or 12-Pack configurations; also available in 4-Pack or 12-Pack Diversity Kits.



Download PDF:



[Read More](#)



## PRESTAN CHILD MANIKINS

**SKU:** PP-CM-100M-MS




The Prestan Professional Child Manikin is designed to look like a child versus an adult with less prominent body structures and softer, more childlike facial features. It is smaller and thinner than the Prestan Professional Adult Manikin. Yet, still realistic to the eye and the touch.

Features:

- Realistic looking and performing manikin is more acceptable to both instructors and students.
- Far easier and faster to carry and set up at training sites.
- Patented face/head tilt simulates how an actual victim's head would move if he required CPR and teaches students how to properly open the airway.
- Easily inserted face-shield/lung-bag provides visual chest rise to ensure students receive the most accurate and realistic training.
- The face shield is secured so it won't slip and there's no need for adhesives.
- Manikins are with the CPR Feedback Monitor
- Prestan Manikins are not made with natural rubber latex.

Download PDF:

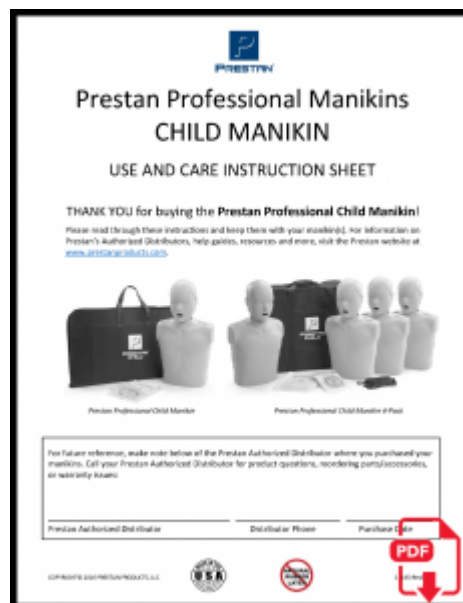

[Read More](#)



## PRESTAN INFANT MANIKIN

**SKU:** PP-IM-100M-MS


Life-like features and weight allows for realistic infant CPR training. Sculpted body contours and realistic skin feel help prepare students for real-life rescues. A head that simulates the way an actual infant's head would move. Face/head tilt enhances the CPR learning experience and teaches students how to open the airway during CPR. Available in single or 4 packs.

Features:

- Realistic looking and performing manikin is more acceptable to both instructors and students.
- Far easier and faster to carry and set up at training sites.
- Patented face/head tilt simulates how an actual victim's head would move if he required CPR and teaches students how to properly open the airway.
- Easily inserted face-shield/lung-bag provides visual chest rise to ensure students receive the most accurate and realistic training.
- The face shield is secured so it won't slip and there's no need for adhesives.
- Manikins are available with the CPR Feedback Monitor - Monitors may also be purchased separately to easily install into non-monitored Prestan Professional Manikins.
- Prestan Manikins are not made with natural rubber latex.



Download PDF:

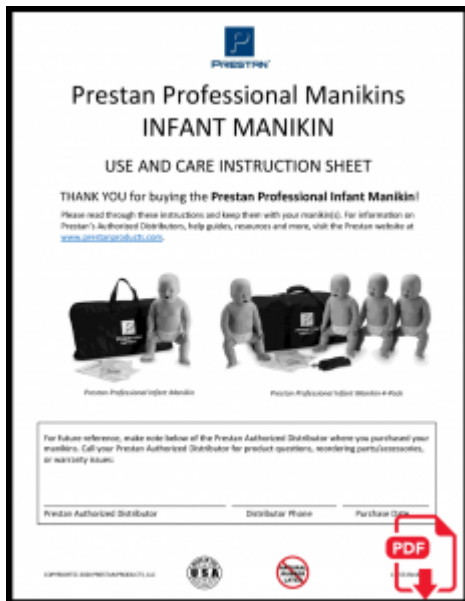

[Read More](#)



## PRESTAN ULTRALITE® MANIKINS


**SKU:** PP-ULM-100-MS

This is PRESTAN's most portable CPR manikin yet! It is amazingly easy to set up and sold in a convenient and lightweight package for efficient training-on-the-go. This durable vinyl and latex-free manikin offer an affordable tool for CPR training with all the quality and realism you expect from PRESTAN.

Features:

- Uniquely fast and easy to set-up.
- Stackable torso and head generate a compact package for transporting.
- Clamshell torso opening provides quick access for the easy-to-insert lung bag.
- Realistic skin resists dirt, grime, and grease.
- Adequate upper chest sizing facilitates AED pad placement and training.
- Visual anatomy landmarks reinforce proper hand placement for compressions.
- Head-tilt illustrates how to open the airway and allows each student to observe a true chest rise when the proper head-tilt chin-lift technique is used.
- Life tested to a half-million compressions.
- The manikin is easy to clean.

Download PDF:

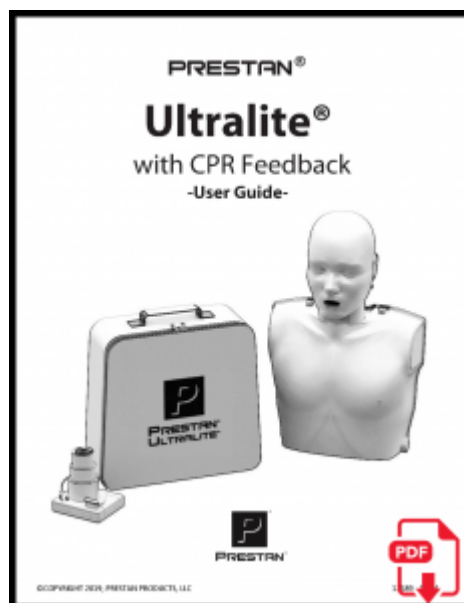

[Read More](#)



## PRESTAN COLLECTION CPR MANIKINS

SKU: PP-FM-300M-MS

The Prestan Professional Training Manikin series is unlike any other on the market. Uniquely designed for realistic experiences, they are available individually or in sets of four for group training. The manikins are designed to match the real-life CPR experience.


In CPR training, a precise feedback indicator is vital. Prestan's revolutionary CPR Rate Monitor provides instant feedback to both instructor and student on the correct compression depth. It enables each student to individually gauge their compression rate as well as allowing the instructor to monitor several students simultaneously. It is equipped with a unique Clicker Mechanism that gives an alert sound when the correct compression depth is reached. This helps ensure the students are using the correct force when administering CPR compressions.




Features:

- CPR feedback monitor that meets 2015 AHA CPR guidelines for Adult Manikin.
- Available in Adult, Child, and Infant models.
- Realistic looking and performing manikin is more acceptable to both instructors and students.
- Far easier and faster to carry and set up at training sites.
- Patented face/head tilt simulates how an actual victim's head would move if he required CPR and teaches students how to properly open the airway.
- Easily inserted face-shield/lung-bag provides visual chest rise to ensure students receive the most accurate and realistic training.
- The face shield is secured so it won't slip and there's no need for adhesives.
- Prestan Manikins are not made with natural rubber latex.

Download PDF:

  
**Prestan Professional Manikins**  
**ADULT MANIKIN**  
USE AND CARE INSTRUCTION SHEET

**THANK YOU for buying the Prestan Professional Adult Manikin!**  
Please read through these instructions and keep them with your manikin(s). For information on Prestan's Authorised Distributors, help guides, resources and more, visit the Prestan website at [www.prestanproducts.com](http://www.prestanproducts.com)





Prestan Professional Adult Manikin      Prestan Professional Adult Manikin 4-Pkg

For future reference, make note below of the Prestan Authorised Distributor where you purchased your manikin. Call your Prestan Authorised Distributor for product questions, ordering, purchase amounts, or warranty issues:


Prestan Authorised Distributor	Distributor Phone	Purchase Date
--------------------------------	-------------------	---------------

©PRESTAN 2016 PRESTAN PRODUCTS, LLC



  
**Prestan Professional Manikins**  
**CHILD MANIKIN**  
USE AND CARE INSTRUCTION SHEET

**THANK YOU for buying the Prestan Professional Child Manikin!**  
Please read through these instructions and keep them with your manikin(s). For information on Prestan's Authorised Distributors, help guides, resources and more, visit the Prestan website at [www.prestanproducts.com](http://www.prestanproducts.com)




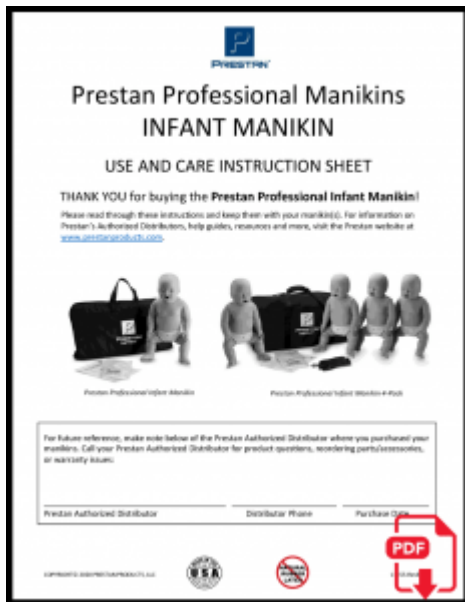
Prestan Professional Child Manikin      Prestan Professional Child Manikin 4-Pkg

For future reference, make note below of the Prestan Authorised Distributor where you purchased your manikin. Call your Prestan Authorised Distributor for product questions, ordering, purchase amounts, or warranty issues:

Prestan Authorised Distributor	Distributor Phone	Purchase Date
--------------------------------	-------------------	---------------

©PRESTAN 2016 PRESTAN PRODUCTS, LLC





[Read More](#)



## PRESTAN FAMILY PACK CPR MANIKINS

SKU: PP-FM-500M-MS




The Prestan Professional Training Manikin series is unlike any other on the market. Uniquely designed for realistic experiences, they are available individually or in sets of four for group training. The manikins are designed to match the real-life CPR experience.


In CPR training, a precise feedback indicator is vital. Prestan's revolutionary CPR Rate Monitor provides instant feedback to both instructor and student on the correct compression depth. It enables each student to individually gauge their compression rate as well as allowing the instructor to monitor several students simultaneously. It is equipped with a unique Clicker Mechanism that gives an alert sound when the correct compression depth is reached. This helps ensure the students are using the correct force when administering CPR compressions.

Features:

- CPR feedback monitor that meets the 2015 AHA CPR guidelines for Adult Manikin.
- Available in Adult, Child, and Infant models.
- Realistic looking and performing manikin is more acceptable to both instructors and students.
- Far easier and faster to carry and set up at training sites.
- Patented face/head tilt simulates how an actual victim's head would move if he required CPR and teaches students how to properly open the airway.
- Easily inserted face-shield/lung-bag provides visual chest rise to ensure students receive the most accurate and realistic training.
- The face shield is secured so it won't slip and there's no need for adhesives.
- Prestan Manikins are not made with natural rubber latex.

Download PDF:

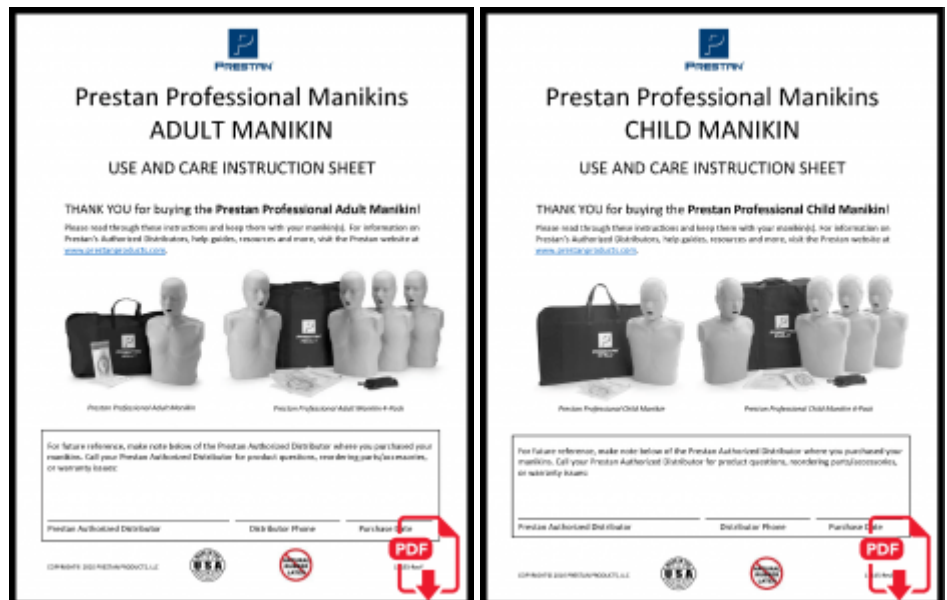

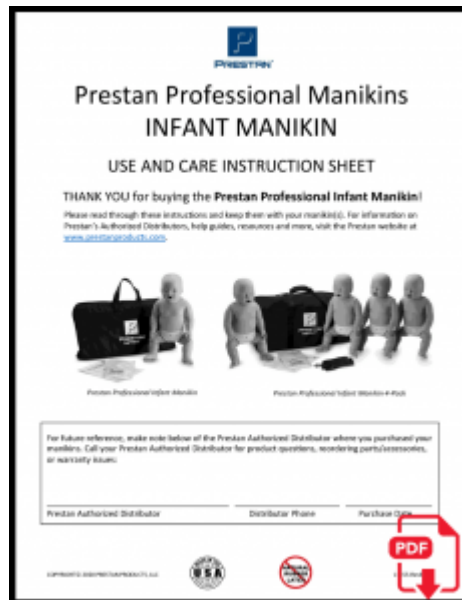

[Read More](#)



## PRESTAN ULTRALITE MANIKINS 4-PACK

**SKU:** PP-ULM-400-MS

The Prestan Professional Training Manikin series is unlike any other on the market. Uniquely designed for realistic experiences, they are available individually or in sets of four for group training. The manikins are designed to match the real-life CPR experience.


The Ultralite Manikin is amazingly easy to set up and is sold in a convenient and lightweight 4-pack for efficient training-on-the-go. This durable manikin offers an affordable tool for CPR training, with all the quality and realism you expect from Prestan.

Features:

- Stackable torsos and heads generate a compact package for transporting multiple units.
- The anatomically correct design includes the xiphoid process.
- Life tested to a half-million compressions.
- Visible chest rise with lung bag use.
- Compatible with AED Trainer pads.



Download PDF:

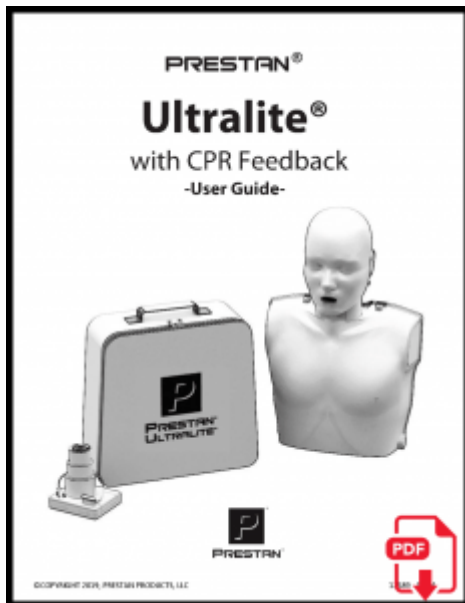

[Read More](#)



## PRESTAN ADULT MANIKINS 4-PACK

**SKU:** PP-AM-400M-MS



The Prestan Professional Training Manikin series is unlike any other on the market. Uniquely designed for realistic experiences, they are available individually or in sets of four for group training. The manikins are designed to match the real-life CPR experience.

Realistic to the eye and the touch, the Prestan adult professional manikin is uniquely designed as a clamshell that accommodates an easy-to-insert face shield/lung bag.

Prestan's revolutionary new CPR Rate Monitor allows for instant feedback to both instructor and student regarding the rate of chest compression. This allows each student to gauge their rate of compressions on their own as well as allowing the instructor to monitor several students quickly and easily.


And with our patented face/head tilt, the Prestan Professional Adult Manikin simulates the way an actual victim's head would move if he required CPR.

In addition, the unique construction of our manikins incorporates a mechanism to help them use the correct force to compress the chest to the correct depth. It comes with 50 A/C face-shield lung bags and a nylon carrying case.

Features:

- Life-like features and weight allows for realistic adult/child CPR training
- Sculpted body contours and realistic skin feel help prepare students for real-life rescues
- A unique clamshell design that accommodates an easy-to-insert face shield lung bag
- CPR Rate Monitor allows for instant feedback to both instructor and student regarding the rate of chest compression
- Includes 4 Manikin with CPR Monitor, Face-Shield Lung Bags, and Nylon Carrying Case.

Download PDF:

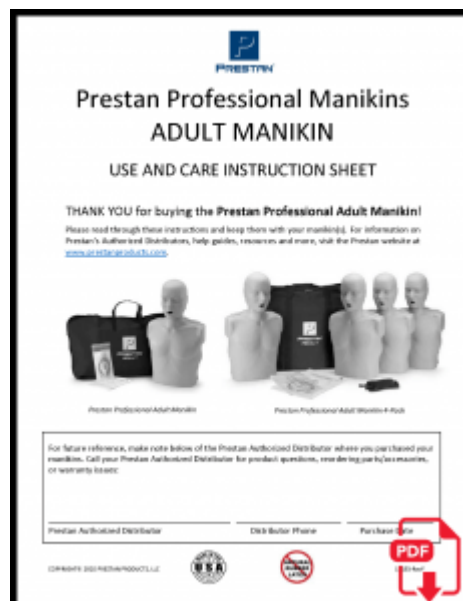

[Read More](#)





## PRESTAN INFANT MANIKINS 4-PACK

**SKU:** PP-IM-400M-MS

This AHA Compliant, 4-Pack of Prestan Infant Manikins-Medium Tone comes equipped with Compression Depth Clicker Feedback Devices and Compression Rate Monitor Feedback Devices.

This 4-Pack of Manikins will optimize your training program by ensuring the student's Compression Depth and Compression Rate are reached with an audible 'clicking' sound and LED indicators provided by 2 separate feedback devices. Additional Items Included: 50 Infant Lung Bags, an Instruction Sheet, and a nylon carrying case.

Achieve this with Prestan Professional's revolutionary new infant CPR manikin with Rate Monitor. The LED indicators allow for instant feedback to both instructors and students regarding their rate of chest compressions. Instructors can monitor several students quickly and easily, while students themselves can visually gauge their compression rate reinforcing more practice time using proper technique.


The depth of compression is a crucial skill to learn as well. Class participants learn proper compression depth with the manikin's audible 'clicking' sound. When compressions are done to the ECC recommended depth, the manikin's 'click' lets students know they are on target. Compression rate and depth - two vitally important components to learning for successful CPR. With Prestan's visual and audible reinforcement of good CPR techniques, students will confidently embrace their role in the cardiac chain of survival.

Features:

- LED indicators on the infant's diaper allow for instant feedback to both instructor and student regarding the accuracy of chest compressions.
- The realistically sized 100% Latex Free infant has a pinch-able nose. "Flick their feet" reminds students to check for the level of consciousness - easily taught since the manikin has fully formed legs with feet. Moveable arms even allow for brachial pulse checks.
- Practice conscious choking back blows on the infants' padded upper back portion.
- The easy to insert lung-bag speeds manikin setup.
- The face-shield is secured so it won't slip and there's no need for adhesives.
- Realistic skin resists dirt, grime, grease, and even adhesive build up from AED electrodes!
- Visual anatomy 'landmarks' such as clavicles, an accurate nipple line, sternum, and xiphoid process facilitates the practice of choking skills as well as reinforcing proper finger placement for compressions.
- A tiltable head illustrates how to open the airway and allows each student to observe a true chest rise when a proper head-tilt chin-lift technique is used.
- The infant's head structure is capable of demonstrating the jaw-thrust maneuver.
- Audible clicker mechanism increases the student's confidence in administering adequate depth of compressions.
- Manikins are capable of being cleaned and decontaminated per CDC guidelines.



Download PDF:

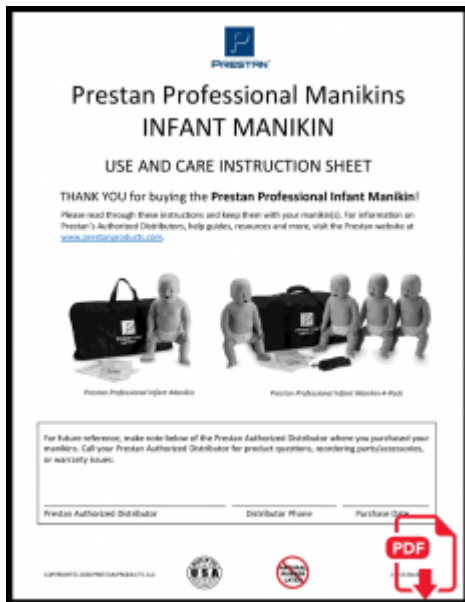

[Read More](#)



## PRESTAN INSTRUCTOR KIT

**SKU:** PP-INSTRKIT-1-MSDS

The PRESTAN Instructor Kit combines our most popular CPR training manikins (two Professional Adults w/ Female Accessory & two Professional Infants) along with two AED UltraTrainers® and a collection of the most commonly used CPR Training Supplies in one single collection, packaged in a heavy-duty canvas bag with wheels & telescoping handle for enhanced portability.

Fully compliant with current industry guidelines, including the most recent AHA Integrated Feedback Directive.

Available as a Diversity Kit (two medium skin tone and two dark skin tone manikins) and with CPR Feedback only.

All PRESTAN CPR Training Manikins are Made in the USA.

## ADDITIONAL DETAILS:

### PRESTAN Instructor Kit Detailed Product Contents

- (2) Professional Adult Manikin w/ CPR feedback (1 Medium Skin & 1 Dark Skin)
- (2) Professional Infant Manikin w/ CPR feedback (1 Medium Skin & 1 Dark Skin)
- (2) AED UltraTrainer® (English/Spanish)
- (2) Female Accessory (1 Medium Skin & 1 Dark Skin)
- (2) Adult CPR Manikin Training Shirt
- (2) Infant CPR Manikin Training Shirt
- (50) Adult Face-Shield/Lung-Bag
- (50) Infant Face-Shield/Lung-Bag
- (2) Adult Bag Valve Mask
- (2) Infant Bag Valve Mask
- (4) PRESTAN Kneeling Pad
- (1) Carry Bag w/ Roller Wheels & Telescopic Handle

### PRESTAN Professional Manikin CPR Rate Monitor

All PRESTAN Professional Manikins are available with a revolutionary CPR Rate Monitor that allows for instant feedback to both instructors and students. The visual feedback from the CPR Rate Monitor gives students a real life feel of delivering 100-120 compressions per minute. Lights continually signal rate progression, confirming compressions are within the 100- 120 cpm range recommended by the Industry Guidelines. The yellow light will blink if the 120 cpm upper limit is exceeded, warning students to slow down. Instructors can monitor several students quickly and easily.

In addition to the visual CPR Rate Monitor, students will also hear a clicker sound as the chest is pushed to the appropriate depth of 2.0 -2.4 inches allowing them to experience the true force needed to deliver real life chest compressions.


The CPR Rate Monitor may be purchased separately to upgrade any non-monitored PRESTAN Professional Manikin.




<b>Sleep Mode/No Power</b>	○ ○ ○ ○
<b>1<sup>st</sup> Full Compression</b> (Wake mode/LED Test Mode)	● ● ● ●
<b>CPM* &lt; 60</b>	○ ● ○ ○
<b>CPM* 60 – 79</b>	○ ○ ● ○
<b>CPM* 80 – 99</b>	● ○ ○ ○
<b>CPM* 100 - 120</b>	● ○ ○ ●
<b>CPM* &gt; 120</b>	● ○ ● ●
<b>Compressions Stopped</b>	○ ● ○ ○

\*CPM = Compressions per minute


⊙ = Flashing

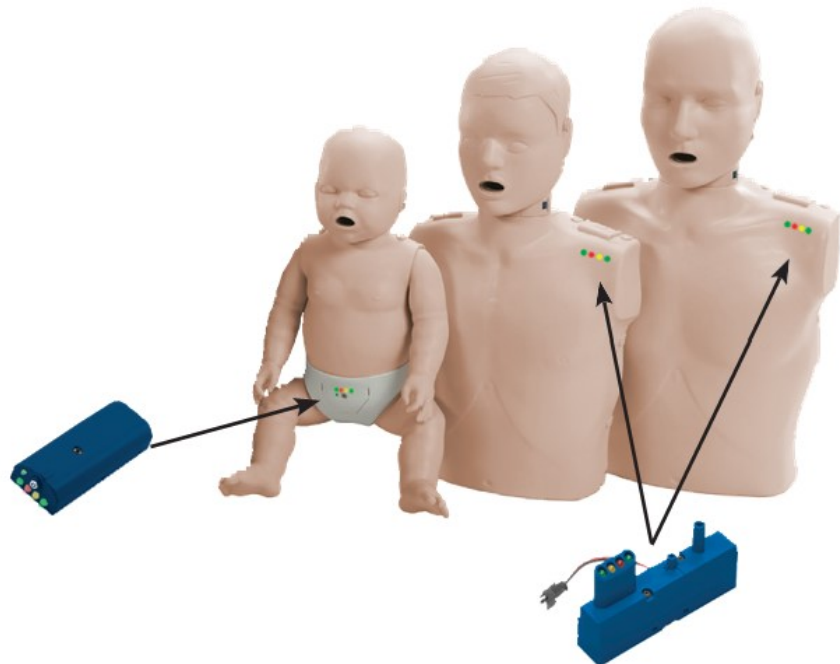

[Download PDF:](#)

  
**Prestan Professional Manikins**  
**ADULT MANIKIN**  
USE AND CARE INSTRUCTION SHEET




**THANK YOU** for buying the **Prestan Professional Adult Manikin!**  
Please read through these instructions and keep them with your manikin(s). For information on Prestan's Authorized Distributors, help guides, resources and more, visit the Prestan website at [www.prestanproducts.com](http://www.prestanproducts.com).




Prestan Professional Adult Manikin      Prestan Professional Adult Manikin 4-Pak


For future reference, make note below of the Prestan Authorized Distributor where you purchased your manikin(s). Call your Prestan Authorized Distributor for product questions, ordering parts/accessories, or warranty issues.

Prestan Authorized Distributor      Distributor Phone      Purchase Date

© 2016 PRESTAN PRODUCTS, LLC                  

  
**Prestan Professional Manikins**  
**INFANT MANIKIN**  
USE AND CARE INSTRUCTION SHEET




**THANK YOU** for buying the **Prestan Professional Infant Manikin!**  
Please read through these instructions and keep them with your manikin(s). For information on Prestan's Authorized Distributors, help guides, resources and more, visit the Prestan website at [www.prestanproducts.com](http://www.prestanproducts.com).



Prestan Professional Infant Manikin      Prestan Professional Infant Manikin 4-Pak

For future reference, make note below of the Prestan Authorized Distributor where you purchased your manikin(s). Call your Prestan Authorized Distributor for product questions, ordering parts/accessories, or warranty issues.

Prestan Authorized Distributor      Distributor Phone      Purchase Date

© 2016 PRESTAN PRODUCTS, LLC                  

**Prestan®**  
**AED UltraTrainer™**






[Read More](#)



## PRESTAN PROFESSIONAL ADULT 50PCS FACE-SHIELD / LUNG BAG

**SKU:** PP-ALB-50

Manikin Replacement Lung Bags

Features:

- Prestan Manikin Replacement Lung Bags.
- Clear plastic inflatable lungs.
- Easy to insert.
- Disposable.

[Read More](#)





## PRESTAN PROFESSIONAL CHILD 50PCS FACE-SHIELD

**SKU:** PP-CFS-50

In classes where there is more than one student per manikin, there is a need for a balance between protecting the student and giving the student a realistic experience.

This is a 50 pack of Filtered Face Shields for Prestan Professional CPR Manikins. If manikins are shared during your training class, you'll want to use a filtered face shield to minimize the risk of cross-contamination between students. The Prestan face shield makes it easy and since it's secured to the manikin, it won't slip. No need for adhesives! Your students will spend less time distracted by an unruly barrier and more time practicing their skills.

Every CPR/AED instructor wants the best outcome when and if the student has a real-life experience giving CPR. This face shield, placed over the manikin's face, allows each student to practice CPR, be protected, and have the benefit of a realistic experience of a full chest rise.

50 pack face shields for Prestan Child Manikins.

Features:

- Professional Child Face Shields by Prestan Products (50 Pack)
- Not made with natural rubber latex

[Read More](#)



## PRESTAN PROFESSIONAL CHILD 50PCS FACE-SHIELD / LUNG BAG


**SKU:** PP-CLB-50

Replacement for Manikin Face Shield Lung Bags.

Features:

- Prestan Manikin Replacement Face Shield Lung Bags
- Clear plastic inflatable lungs
- Easy to insert
- Disposable

[Read More](#)

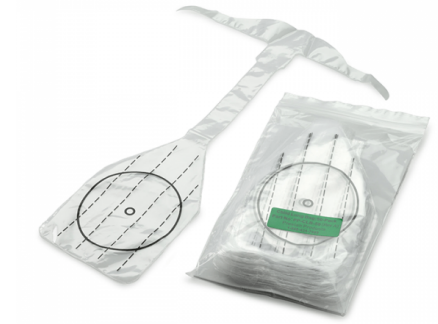

## PRESTAN PROFESSIONAL INFANT 50PCS FACE-SHIELD / LUNG BAG

**SKU:** PP-ILB-50

Replacement for Manikin Face Shield Lung Bags

Features:

- Prestan Manikin Replacement Face Shield Lung Bags
- Clear plastic inflatable lungs
- Easy to insert
- Disposable

[Read More](#)



## PRESTAN PROFESSIONAL ULTRALITE 50PCS FACE-SHIELD / LUNG BAG

**SKU:** PP-ULB-50

The face shield lung bag delivers a realistic experience of a full chest rise during CPR training. These lung bags fit the Prestan Ultralite series CPR manikins only.

This pack includes 50 lung bags. Insertion of the one-piece lung bag design is a breeze! Fold it up, slide it in and you are ready for training.

Features:

- Easy to insert
- Work with Prestan Ultralite CPR Manikins
- No tools necessary to insert

[Read More](#)



## PRESTAN FEMALE ACCESSORY


**SKU:** PP-ULB-50-1

The PRESTAN Female Accessory™ (*patent-pending*) has been designed to build confidence in CPR training, improve CPR survival rates in women, encourage dialogue & breakdown barriers, and eliminate gender bias in CPR administration.

Securely fits both the PRESTAN Professional Adult/Series 2000 & Ultralite Manikins. Pair with the PRESTAN CPR Training Shirts for a full real-life training experience.

Available in packs of 4 and in medium or dark skin tones.

[Read More](#)

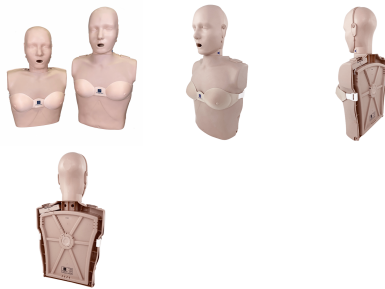

## PRESTAN MASKS ADAPTERS

SKU: 10076-PPA

Use this affordable, disposable adapter in place of expensive mask valves for cost-effective training. It features an integral air filter that obstructs the flow of contaminants. Fits all standard rescue masks (15mm I.D. or 22mm O.D.).

**\*For training purposes only.**

Features:

- It features an integral air filter that obstructs the flow of contaminants.
- Fits all standard rescue masks
- Rescue mask not included

[Read More](#)



